



Micro-Scientific™ Cuts-It® Instrument Spray is specially formulated to safely remove blood, fat, tissue and body fluids from surgical instruments and equipment. The wetting agents will keep instruments moist for hours allowing for safer and easier processing in decontamination.

- **Immediately dissolves blood, fat and other forms of bioload.**
- **May be used in conjunction with other detergents and cleaning equipment.**
- **Product safe and compatible with all metals, plastic, glassware and rubber.**

Instrument Spray

Order No.	Size	Physical Properties	
M60022	24 fl. oz. (710 mL)	Form:	Liquid
		Color:	Yellow
		Odor:	Fresh
		pH Concentration:	8.7

Uses	Temperature
<input checked="" type="checkbox"/> Presoak <input type="checkbox"/> Automatic Washer <input type="checkbox"/> Manual <input type="checkbox"/> Automatic Endoscope Reprocessor <input type="checkbox"/> Ultrasonic <input type="checkbox"/> Cart Washer	Usage: Room Temperature Storage: Cool, dry area

Product Features	Consumer Benefits
Rapid and Effective:	Immediately dissolves blood, fat and other forms of bioload.
Versatile:	May be used in conjunction with other detergents and cleaning equipment.
Rinsing Agents:	Rinses spot & residue-free.
Neutral pH:	Product safe and compatible with all metals, plastic, glassware and rubber.
Environmentally Friendly:	Biodegradable, phosphate-free formula.
Triple Surfactant System:	Removes blood and fats while preventing hard water deposits.

Procedures	Directions for Use
Directions:	Wear appropriate personal protective equipment based on safe work practices. Immediately after surgery, or prior to the first processing step, instruments are sprayed directly with Cuts-It® on soiled areas. Pre-spraying all organic bioload, such as blood and tissue, will allow for safer and easier processing.