



Opti-Cide® MAX cleans and disinfects hard, non-porous surfaces such as acrylic, metals (aluminum, brass, copper, stainless steel, chrome), glass, plastics (polycarbonate, polypropylene, polystyrene and polyvinylchloride (PVC)), Plexiglas®, and vinyl. Opti-Cide® MAX kills hard-to-kill ESKAPE and ESKAPE MDRO organisms with a 1 minute♦ kill time.

- **1 Minute♦ kill time**
- **Kills ESKAPE and ESKAPE MDRO organisms**
- **Bactericidal, Fungicidal, Virucidal, Tuberculocidal**

Order No.	Size
M60034	160 Wipes (6 x 6.75 inches)
M60038	65 Wipes (9 x 12 inches)
M60035	1 Gallon (3.8 L)
M60036	24 fl. oz. (710 mL) Spray

Disinfectant Cleaner

Kills ESKAPE Organisms: 1 Minute contact time

[E] *Enterococcus faecium*
 [S] *Staphylococcus aureus*
 [K] *Klebsiella pneumoniae*
 [A] *Acintobacter baumannii*
 [P] *Pseudomonas aeruginosa*
 [E] *Enterbacter aerogenes*

Kills ESKAPE MDRO Organisms: 1 Minute contact time

[E] *Enterococcus faecium*
 [S] Methicillin-Resistant *Staphylococcus aureus* (MRSA)
 [K] Carbapenum Resistant *Klebsiella pneumoniae* NDM
 [A] Multi-Drug Resistant *Acintobacter baumannii*
 [P] *Pseudomonas aeruginosa*
 [E] *Enterbacter cloacae*

Procedures	Solution Directions for Use
Directions:	It is a violation of Federal law to use this product in a manner inconsistent with its labeling.
To Use as a Cleaner and Deodorizer:	Apply solution to surface and allow surface to remain wet for 30 seconds prior to wiping with a paper towel or clean towel/sponge. This product can also be used to pre-clean all surfaces to be disinfected.
To Disinfect Hard, Nonporous, Surfaces/Articles:	Spray solution to thoroughly wet pre-cleaned hard, non-porous surfaces. Allow surface to remain wet for 1 minute♦, prior to wiping or allow surfaces to air dry. Note: Disinfected surfaces which may come in contact with food must be rinsed with potable water after cleaning and/or disinfection and before reuse. ♦This symbol indicates a 2 minute contact time is required to kill TB.

Procedures	Wipes Directions for Use
Directions:	It is a violation of Federal law to use this product in a manner inconsistent with its labeling.
Dispensing Directions:	To start feed, remove cover and discard seal. From the center of the wipe roll, pull up a wipe corner, twist it into a point and thread it through the hole located on the container cover. Pull through about one inch. Replace cover. Pull out first wipe and tear off at an angle. Remaining wipes feed automatically, ready for the next use. When not in use, keep center cap closed to prevent evaporation.
To Disinfect Hard, Non-porous Surfaces/Articles:	Use a fresh wipe to thoroughly wet pre-cleaned hard, non-porous surfaces. Allow surfaces to remain wet for 1 minute♦, prior to wiping or allow surfaces to air dry. Note: Disinfected surfaces which may come in contact with food must be rinsed with potable water after cleaning and/or disinfection and before reuse. ♦This symbol indicates a 2 minute contact time is required to kill TB.