

ENZYCLEAN II

Dual Enzyme Detergent

Enzyclean® II Dual Enzyme Detergent is specially formulated for use as a presoak or manual detergent. The protease and amylase enzymes in Enzyclean® II break down organic and inorganic soils, residues, films and deposits and ensure thorough and complete cleaning, leaving instruments moving freely and spot-free.

FEATURES & BENEFITS

- **Dual Enzyme.** Protease enzyme effectively breaks down proteins. Amylase enzyme breaks down carbohydrates.
- **Rust Inhibitors.** Reduces likelihood of rust, tarnish or corrosion.
- **Rinsing Agents.** Rinses spot & residue-free.
- **Highly Concentrated.** Low cost per use.
- **Neutral pH.** Safe and compatible with all metals.
- **Bacteriostatic.** If stored properly, not favorable to bacterial growth.



ORDER INFORMATION ► Item Code: B9HC • Size: 1 Gallon (3.8 L) • Case: 4 Units/Case

ENZYCLEAN II

Dual Enzyme Detergent



USES Presoak • Manual

PHYSICAL PROPERTIES Form: Liquid • Color: Green • Odor: Spearmint • pH Concentration: 8.75

TEMPERATURE Usage: 60°-140° F (16°-60° C) • Storage: Cool, dry area

DIRECTIONS

Wear appropriate personal protective equipment based on safe work practices.

PRESOAK & MANUAL CLEANING

Add 1/8 fl. oz. (3.7 mL) to 2 fl. oz. (59 mL) per gallon (3.8 L) of warm water. Allow to soak for 2-3 minutes or longer. Discard solution after each use.

Dilutions vary depending on temperature, water quality and equipment. Validated cleaning is recommended.