

# MICROLUBE C

## Instrument Lubricant

MicroLube™ C is specially formulated to be used as a lubricant and rust inhibitor for all types of surgical and dental instruments, including carbon steel. It is compatible with all sterilization methods, and if stored properly, can be reused for 28 days without becoming contaminated.

### FEATURES & BENEFITS

- **Rust Inhibitors.** Reduces likelihood of rust, tarnish or corrosion. Extends life of instruments, resulting in fewer repairs.
- **Neutral pH.** Product safe and compatible with all metals.
- **Non-silicone Lubricant.** Patient-safe lubricant will not cause abscesses in body.
- **Bacteriostatic.** If stored properly, not favorable to bacterial growth.
- **Environmentally-Friendly.** Phosphate-free formula.



**ORDER INFORMATION** ▶ Item Code: T2 • Size: 1 Gallon (3.8 Liters) • Case: 4 Units/Case

# MICROLUBE C

Instrument Lubricant



**USES** Manual • Automatic Washer

**PHYSICAL PROPERTIES** Form: Liquid • Color: White • Odor: None • pH Concentration: 6.3

**TEMPERATURE** Usage: Room Temperature • Storage: Cool, dry area

## DIRECTIONS FOR USE

Wear appropriate personal protective equipment based on safe work practices.

## MANUAL LUBRICATION

Add 1 part lubricant to 6 parts distilled or demineralized water. If mixed lubricant is covered between use, solution can be changed every 28 days; however, if lubricant is used frequently, it is recommended to change the mixture each week. Use instrument lubricant daily to obtain maximum results.

## AUTOMATIC WASHING EQUIPMENT

Add 1/8 fl. oz. (3.7 mL) to 2 fl. oz. (59 mL) per gallon (3.8 L) of water.

**Dilutions vary depending on temperature, water quality and equipment.**