

GOO GONE

Topical and Surface Adhesive Remover

Micro-Scientific™ Goo Gone® Topical and Surface Adhesive Remover removes tape and adhesives completely and safely from instruments, equipment and containers. The unique formula painlessly removes tape and adhesives from skin and is non-irritating.

FEATURES & BENEFITS

- **Safe.** Product is safe and compatible with all metals.
- **Rapid and Effective.** Removes tape and adhesives quickly and effortlessly.
- **Painless.** Does not harm skin while allowing removal of tape and adhesives without discomfort.
- **Highly Concentrated.** Low cost per use.
- **Bacteriostatic.** If stored properly, not favorable to bacterial growth.
- **Environmentally-Friendly.** Biodegradable, phosphate-free formula.



ORDER INFORMATION ▶ Item Code: R6A • Size: 16 fl. oz (472 mL) • Case: 12 Units/Case

GOO GONE

Topical and Surface Adhesive Remover



USES Manual

PHYSICAL PROPERTIES Form: Liquid • Color: Blue • Odor: Citrus • pH Concentration: n.a.

TEMPERATURE Usage: Room Temperature • Storage: Cool, dry area

DIRECTIONS

Saturate cotton swab or gauze with Micro-Scientific™ Goo Gone® Topical and Surface Adhesive Remover and apply it to the edge of the tape or adhesive. Pull the tape gently and keep applying Adhesive Remover. Use more Adhesive Remover if tape is being removed from hirsute areas. Wash skin with soap and water after use then rinse and dry with clean cloth. After using Adhesive Remover on instruments, equipment or containers, follow complete processing procedures again before proceeding with disinfection or sterilization.